

Anatomy: **ABDOMEN** - Exams
 Sub-Anatomy: **Non-Contrast MRCP** - Routine Coil: **SENSE XL TORSO**

| SEQUENCE - BASICS | | | | VARIABLE PARAMETERS | | | | | | | | | | | | | | |
|---------------------|----------------|------------------|----------------|---------------------|----------|--------|--------|-----------------|----|----|-------|-------------|----|------------------|--------------|-----------------|--------------------|--|
| PLANE | SEQ | Slice / Gap (mm) | Misc / Comment | MTX | FOV (cm) | % RFOV | Scan % | Pixel size (mm) | TR | TE | FA/TI | Nex Avg Acq | NS | ETL Turbo Factor | Phase Encode | Scan TIME (min) | Pixel Shift BW-kHz | |
| ROUTINE | | | | | | | | | | | | | | | | | | |
| COR | 2D T2 SSFSE | 5/1 | BH | | | | | | | | | | | | | | | |
| AX | 2D T2 SSFSE | 5/1 | BH | | | | | | | | | | | | | | | |
| AX | 2D T2 FS SSFSE | 5/1 | BH | | | | | | | | | | | | | | | |
| AX | 2D T1 IP/OP | 5/1 | BH | | | | | | | | | | | | | | | |
| AX | 2D DWI FS | 7/1 | b=0,50,400,800 | | | | | | | | | | | | | | | |
| AX | 3D mDIXON | 4 -> 2 | Single phase | | | | | | | | | | | | | | | |
| RAD | 2D SShMRCP | 40 | BH | | | | | | | | | | | | | | | |
| COR | 3D SShMRCP | 2 -> 1 | | | | | | | | | | | | | | | | |
| ↓ OPTIONAL ↓ | | | | | | | | | | | | | | | | | | |

ORDERABLE: MR MRCP ABDOMEN WO IV CONTRAST.

TIPS:

